




LUPIN

generichealth

LUPIN GROUP COMPANY

Teriparatide Lupin Educational Material

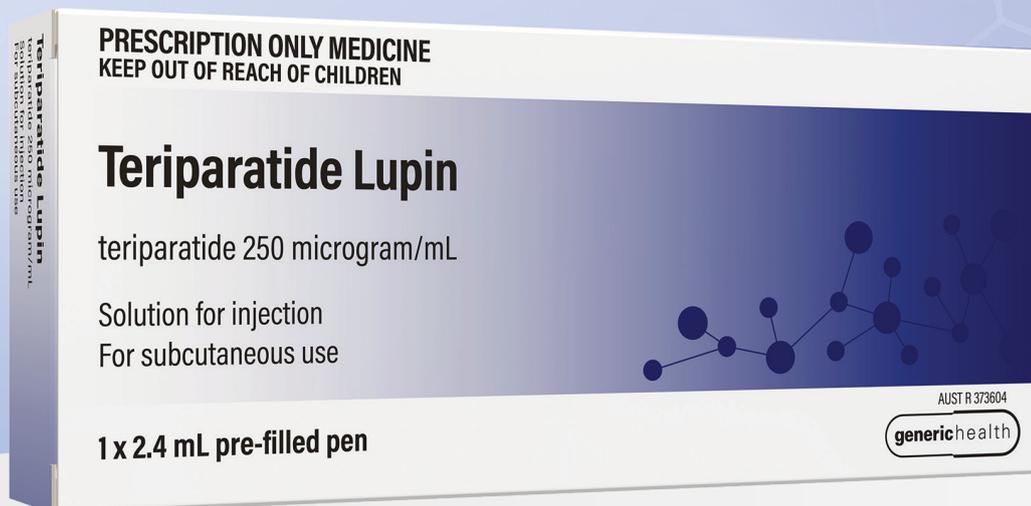

Disclaimer

The educational materials provided in the following pages are intended for patients who have been prescribed a medicinal product supplied by Generic Health by their treating doctors. By reading this booklet, you agree that you are a patient who is currently using or about to use a medicinal product supplied by Generic Health for a condition that has been diagnosed by your treating doctor.

By continuing, you understand that the educational materials are provided herewith to assist you to use Generic Health's product correctly as it is intended.

Please always read the Consumer Medicine Information (CMI), instruction manual or pack insert, as applicable, supplied with your medicine in full before use. For any medical information queries or to report an adverse event, please contact Generic Health on +61 3 9809 7900 or by email at ghinfo@generichealth.com.au


Do not use Teriparatide Lupin if you have previously received teriparatide therapy for a total of 24 months. This total of 24 months therapy may have been made up of several shorter courses of teriparatide. If you are not sure if you have previously received teriparatide, speak to your doctor or pharmacist. **You will be required to sign your consent that you understand the 24-month lifetime limit.** Keep this consent form as you may need to show this to your doctor in the future. Your pharmacist may also ask to see your consent form.



Pack Details

Teriparatide Lupin is supplied in an injection pen.

The pen contains medicine for once-a-day injection for 28 consecutive days.

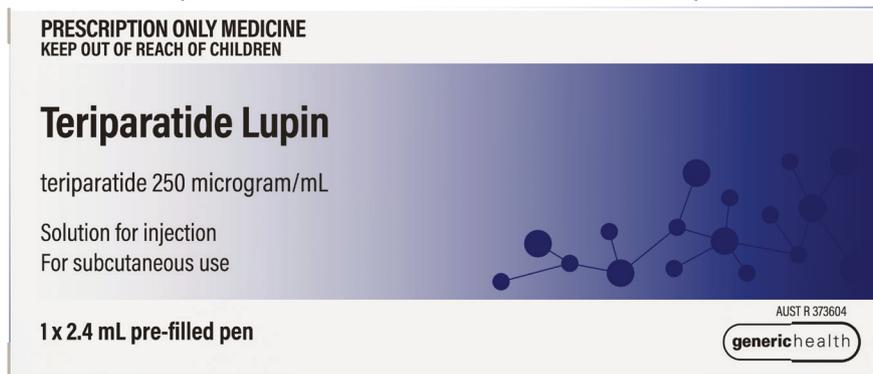

Use a new needle for each injection. Needles are not provided with the pack.

Consult your physician or pharmacist about which needle is most suitable for you.


Refer to section below for the optimal needle sizes to use.

Recommended Needle Size: 31 gauge, 5mm length, CE Certified

If recommended needle size is not available, pen needles that meet the following specifications can also be used: 29-31 gauge; 5 mm to 12.7 mm in length; CE Certified.

Overview

- 1 Pen cap
- 2 Cartridge with medicine
- 3 Label
- 4 Dose window
- 5 Dose knob
- 6 Dose activator
- 7 Large outer cap
- 8 Small inner cap
- 9 Needle
- 10 Seal foil

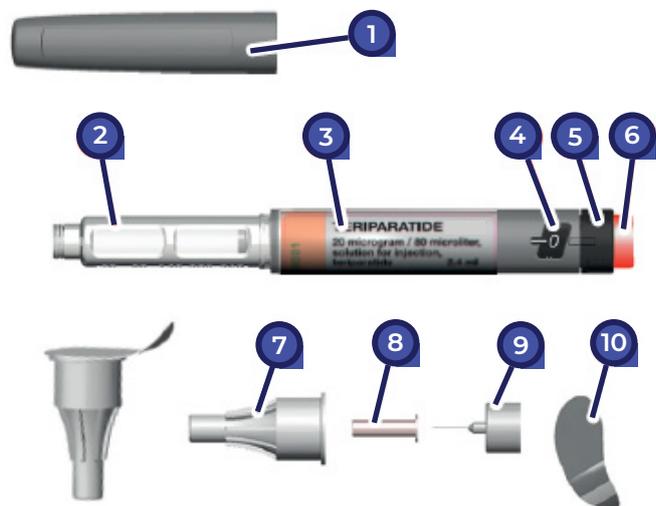

Safety Instructions

Use of pen

CHECKLIST



Check pen label to ensure you use the right medicine.



Check the expiry date.



Check the medicine is clear, colourless & free of particles.

Do not use the pen for more than 28 injections. Note the 1st day & the 28th day of injection.



28

Teriparatide Lupin must be administered using the injection pen provided. Do not transfer to a syringe.



Anyone who is blind/visually impaired must seek help from a support person or carer.



Safety Instructions

Important Information


You will be required to sign a consent form understanding the 24-month lifetime limit.



Read the instructions on the package leaflet provided with the pen.



Use a new sterile needle for each injection. Do not reuse needles.

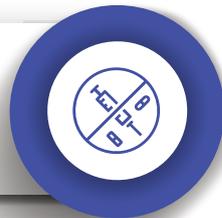

You must not use Teriparatide Lupin if you have previously received the therapy for a total of 24 months.



If you have any questions, contact your physician, pharmacist or caregiver.



Do not share your pen as this may risk transmission of infectious diseases.





User Manual

Preparing the Pen Before the First Injection



1. Before **FIRST** injection, you have to prepare your pen as described here.
2. You have to perform this step only once. You do **NOT** need to repeat this procedure for the second and any following injection.

Attaching the needle (Step 1)




Remove pen cap.

Attaching the needle (Step 2)




Take a new needle and remove the seal foil from the needle cap.

Attaching the needle (Step 3)

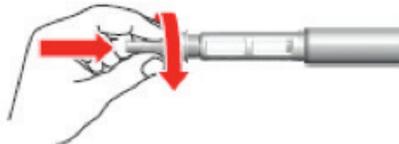


Attach needle with large outer cap to the pen. Screw large outer cap clockwise until hard stop.

Attaching the needle (Step 4)

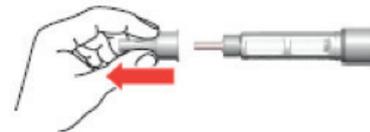


Remove large outer cap from the needle and keep it for later use (Refer Step 9).

Dialling the dose (Step 5)

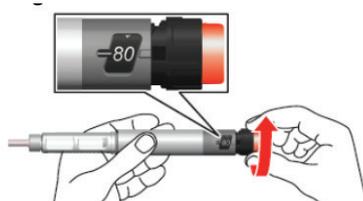

Turn the dose knob to the hard stop. Make sure the figure "80" is fully visible and centred in the dose window with the white mark aligned in the window notch.

Dialling the dose (Step 6)

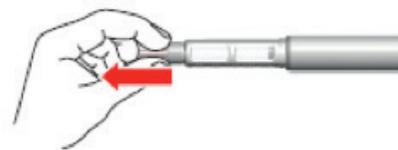


Remove the small inner cap from the needle and dispose it.

Dialling the dose (Step 7)



Hold pen with needle up. Push the dose activator to the hard stop and hold for 5 seconds. Collect the expelled liquid with a tissue.

Dialling the dose (Step 8)

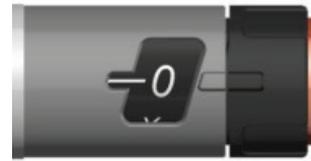


Confirm dose. Make sure the figure "0" is fully visible and centred in the dose window, the white mark is aligned in the window notch and the debossed marks of dose knob and pen body are aligned.

Removing the Needle

Remove needle after preparation of pen, as it poses a risk of being contaminated.

Removing the Needle (Step 9)

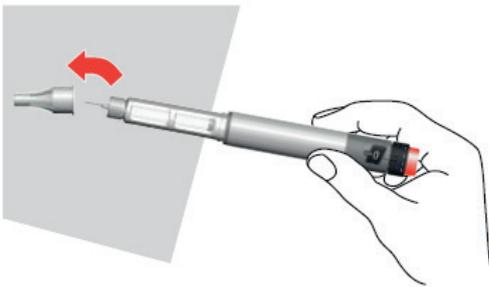


Carefully push needle into the large outer cap kept aside before in Step 4. To prevent injury, do not touch the needle directly.

Removing the Needle (Step 10)




Unscrew needle by turning the large outer cap counter clockwise and pull needle from pen.

Removing the Needle (Step 11)

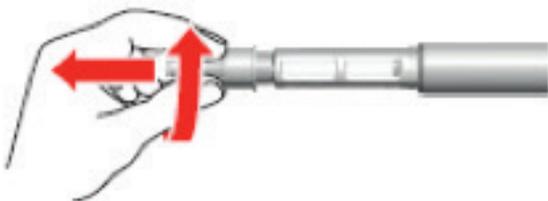

Dispose of the needle in a puncture safe container which can be obtained from your pharmacy or from your caregiver.

Removing the Needle (Step 12)

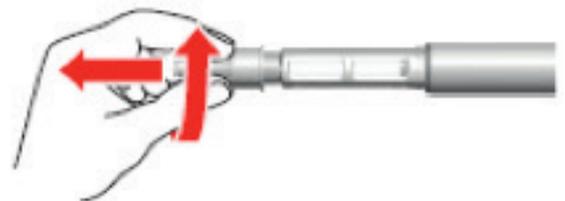

Re-attach pen cap to pen.

Now your pen is prepared for the first and all further injections as described in the following chapter.



User Manual

Injecting Teriparatide Lupin




1. Prepare yourself for injection.
2. Wash your hands before every injection.
3. Prepare the injection site (on thigh or abdomen) as directed by your doctor, pharmacist or caregiver.

Use a new needle for every injection because a new needle is sharp and allows an almost pain-free injection. A used needle poses a risk of being clogged or contaminated.


Follow Steps 1 to 6 on page 4 of this booklet

Dialling the dose (Step 7a)

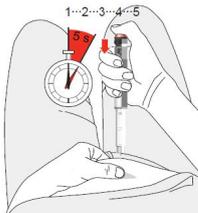

Gently hold a fold of skin from thigh or abdomen.

Dialling the dose (Step 7b)

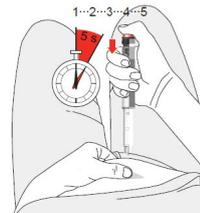


Insert the needle preferably at a 90 degree angle at the prepared injection site. Push the dose activator to the hard stop and hold for 5 seconds. Count up slowly to 5.

Dialling the dose (Step 7c)



Pull out the needle from the skin.

Dialling the dose (Step 8)

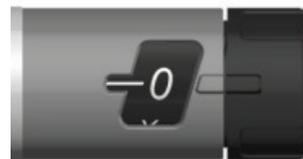

Confirm dose. Make sure the figure "0" is fully visible and centred in the dose window, the white mark is aligned in the window notch and the debossed marks of dose knob and pen body are aligned.

Removing the Needle : Always remove needle immediately after use of pen.

Remove the needle as in Steps 9 to 12 on page 5 of this booklet



Keeping the Injection Diary

Note the current date of injection next to the relevant day, and calculate the date of the 28th injection, if not already done so, in the Injection Diary at the end of these Instructions for Use.




Preventive Care



Store the pen in the refrigerator, preferably in a door compartment. Only take the pen out of the refrigerator for use.




Don't store the pen next to the back wall of the refrigerator or in the freezer. The medicine becomes unusable if frozen.

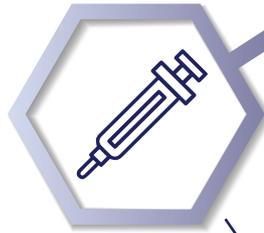

Always close the pen with a pen cap.




Keep pens and needles out of the reach of children.

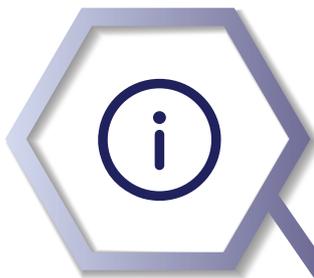

Put the pen back into the refrigerator immediately after you have used it.




The pen must be disposed of, as directed by your physician or pharmacist, on the day of the last injection (as in the Injection Diary).

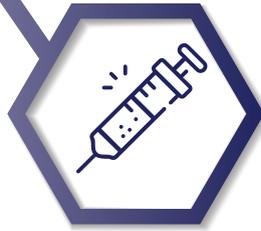

Do not store a pen with a needle attached. Doing this may cause air bubbles to form in the medicine cartridge.



Dispose of pen once all injections have been taken, even if there is solution remaining in the cartridge.

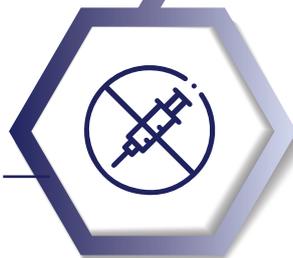

Attach pen cap before disposal. Don't dispose pen with needle attached.

FAQs

What to do if the dose activator is blocked or you have the impression you have not injected the full dose?

Do not give a second injection the same day. Proceed with your regular injection the next day.

What to do if something looks wrong?

- Only use the pen if the solution is clear, colourless & free of particles.
- If you cannot solve the issue by yourself or if you are uncertain, contact your physician, pharmacist, or caregiver.

While preparing the pen for first use no solution is expelled?

Refer to Dialling the Dose (Steps 5-8) under the section - "Preparing the Pen Before the First Injection" on pages 4 & 5 of this booklet.

What if the pen is damaged?

Do not use the pen.

What precautions should be taken if air bubbles are noticed in the cartridge?

You can use your pen without any concern.



LUPIN

generichealth

LUPIN GROUP COMPANY

For further information,
please contact Generic Health Pty Ltd :

P : 03 9809 7900

F : 03 9809 7999

W : www.generichealth.com.au

Generic Health Pty Ltd.
Suite 2 Level 2
19-23 Prospect Street
Box Hill VIC 3128

Prepared in August 2024

**For more resources,
visit our website**

